
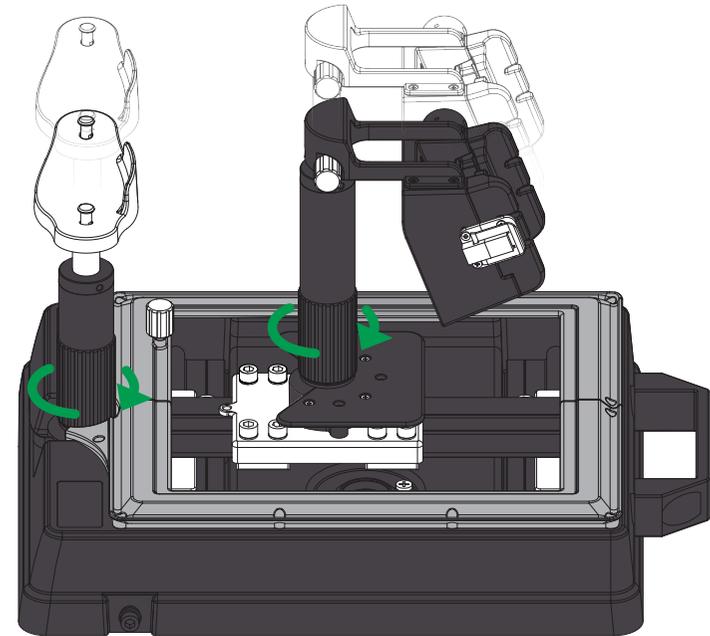
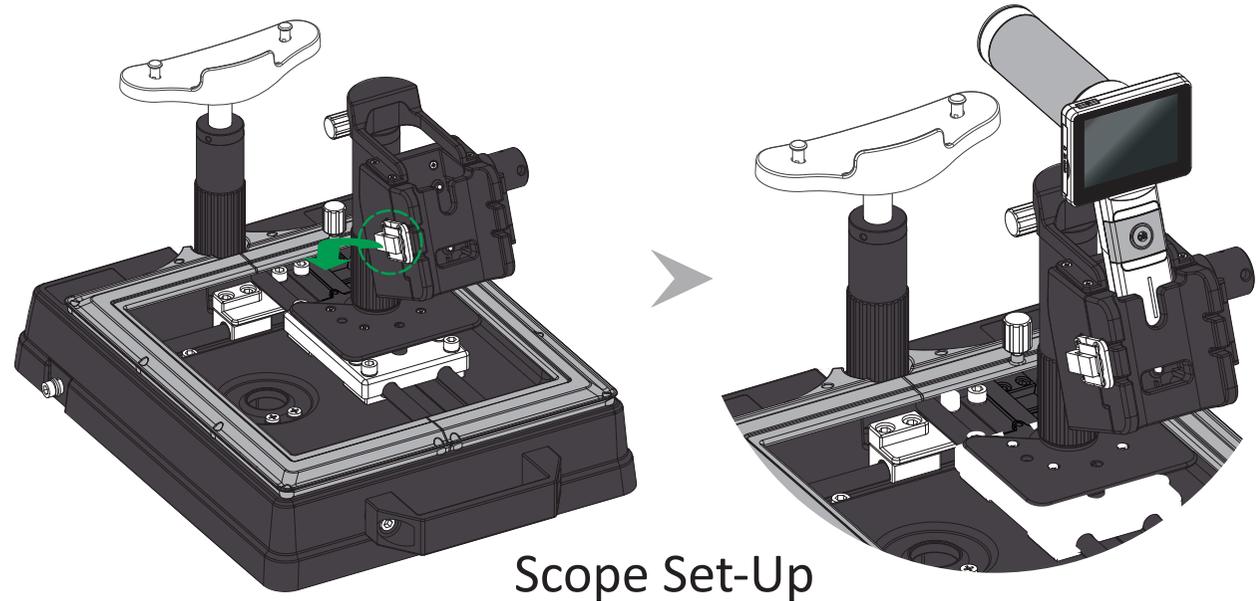


HORUS

CR 100

Portable Chin Rest Assembly Guide

www.miis.com.tw

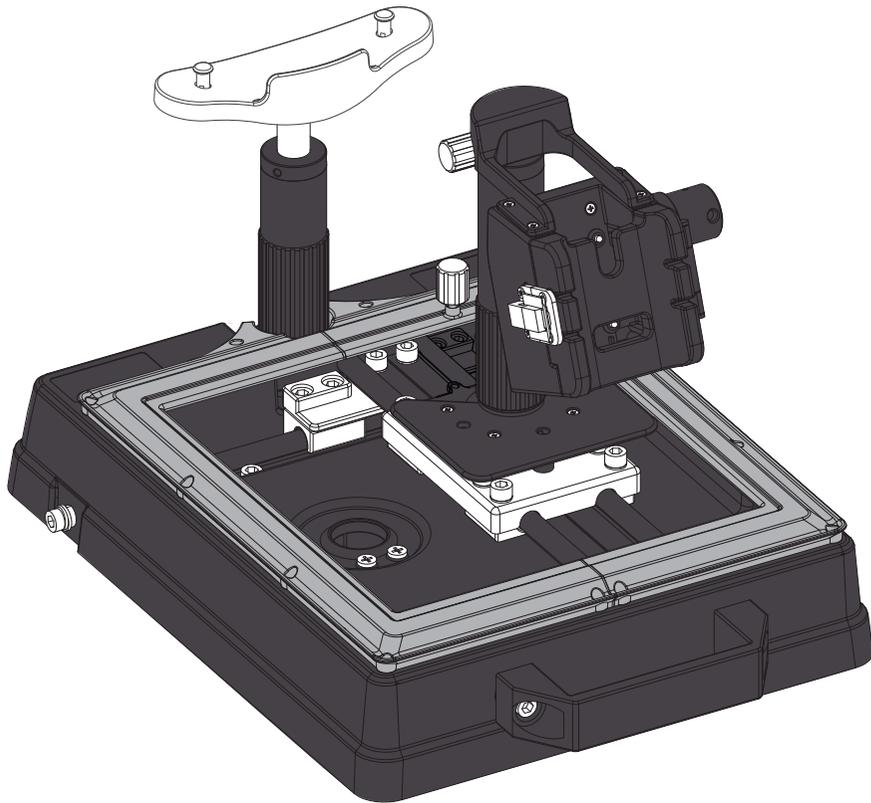

The Height Adjustment of Chin Rest and Scope



DOC. No. 2020.21-B
Copyright©2020 MiiS Inc.
All right reserved.

Symbols



Caution must be taken. Read user's manual before use.



The operator is advised to read the instructions of user's manual.



Date of Manufacture.



Manufacturer.




Use chin rest paper for each new patient.


Ensure that the examinee has not placed his/her hand or fingers under the chin rest. Otherwise, hand or fingers may be hurt.

Chin Rest Set-Up


1. To turn chin rest to the back. To loosen knob anticlockwise, and push latch aside to open back cover.

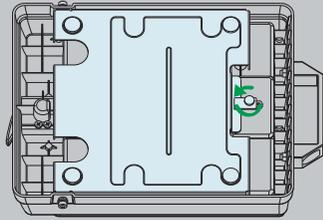


2. To take parts a, b, c out, and release quick-release ring. To close back cover and turn chin rest to the front.

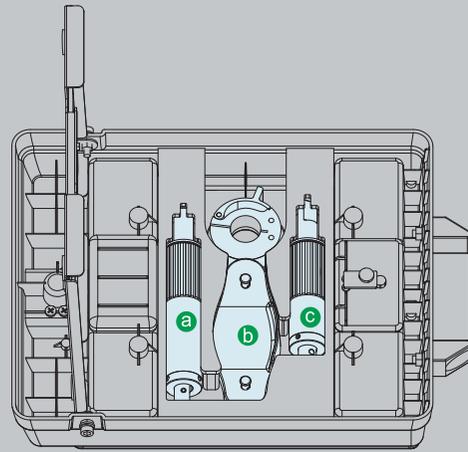

3. To take part d out.

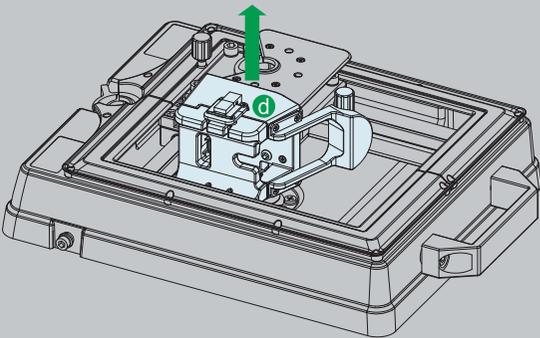


After assembling a and d, to fasten knob clockwise.

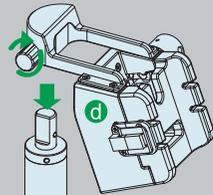

step 1

To assemble b and c.

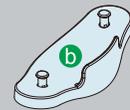


step 1


step 2

step 3

step 4

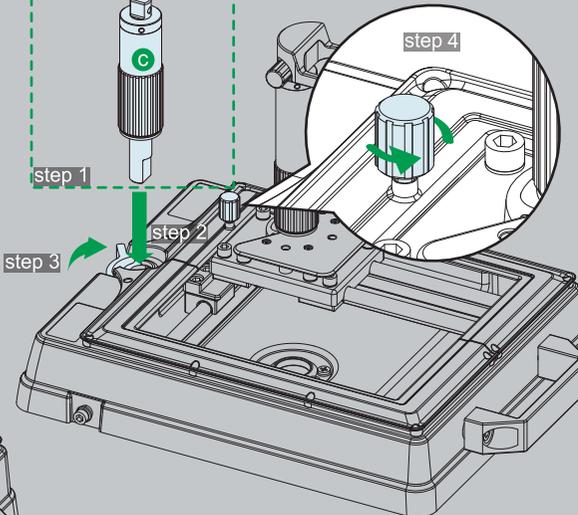


4. To insert assembled a + d (step 1~2), then press the handle (step 3).

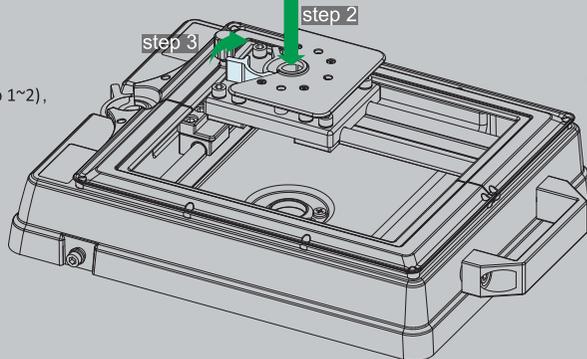



5. To insert assembled b + c (step 1~2), then press the handle (step 3). Next, to loosen the knob anticlockwise to release guide (step 4).

Chin Rest Storage

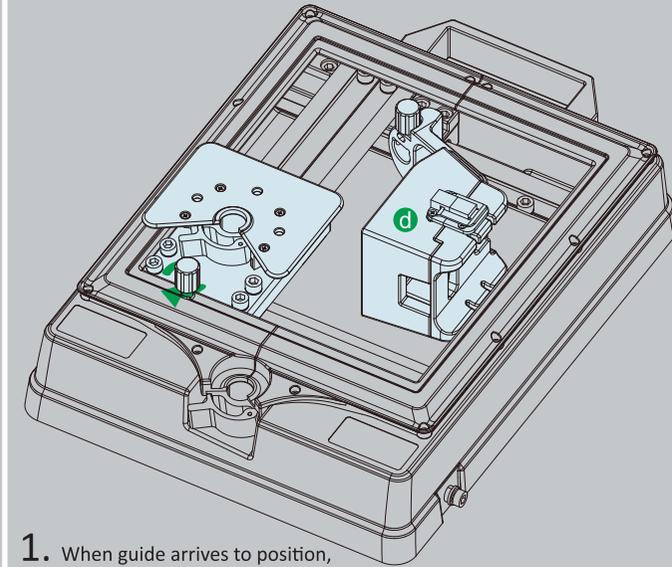

1. When guide arrives to position, press down the handle hard and take parts a, b, c apart and then insert part d.

To press down the handle.

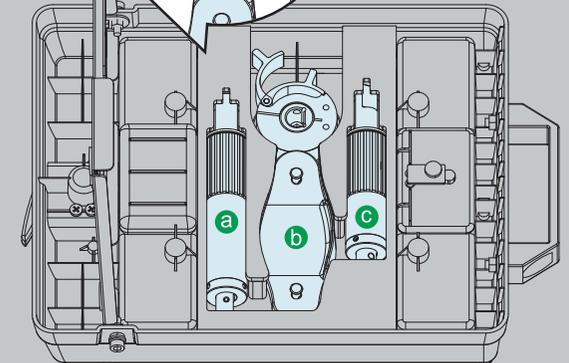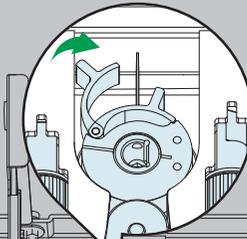

2. To fasten quick-release ring to fix part d. Next, to rotate parts a and c, to their shortest, and then to put a, b, c inside. Finally to close back cover and fasten.