

1. Please make sure that all (6) "Red Points" are inside of the Green Ring.
2. and then "Fine tuning" the test distance until the all "red points" clearly. (PS: Hands cannot shake)



Condition A: Too Far (Please shorten the distance)

Condition B: Too Near (Please make a long distance)

3. it can capture automatically when all "red points" become clearly and listening the Continuous sound of "Di Di..." at the same time. (PS: Actually, you almost got the Data, but you shake the hand at last, pls hold on when you hear the "Di..Di..." — It prove that you almost get the data, when you hear that sound)

PS:

1. Please Blocking the Stray light with hand (It is better measuring in the dark room)



Wrong

Correct

2. Patients cannot to blink frequently on the process of measurement.